


# Mextra® Superabsorbent


Versatile solution contributing to successful management of highly exuding wounds

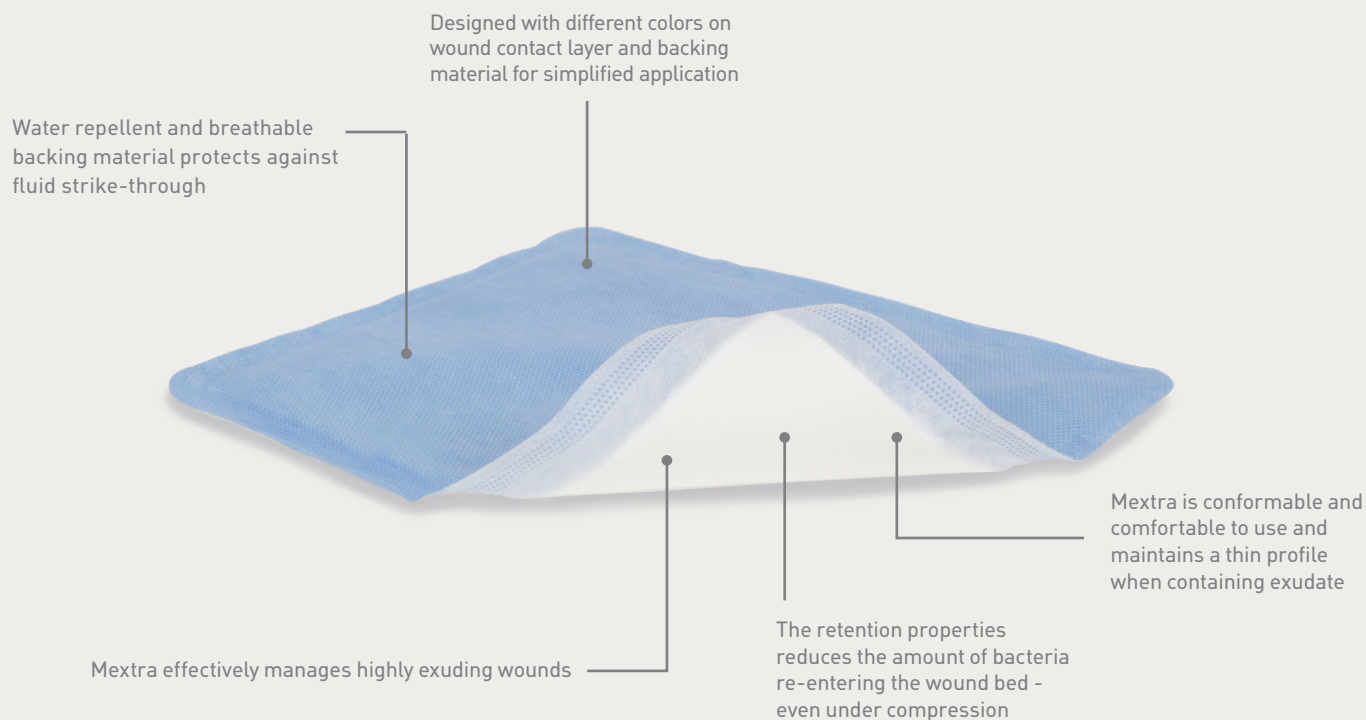


## Key features of Mextra® Superabsorbent:

- Rapid fluid uptake
- High absorption and retention capacity that minimises the risk of maceration
- Reduces the amount of bacteria re-entering the wound bed - even under compression
- Conformable and comfortable to use

## How to use Mextra® Superabsorbent

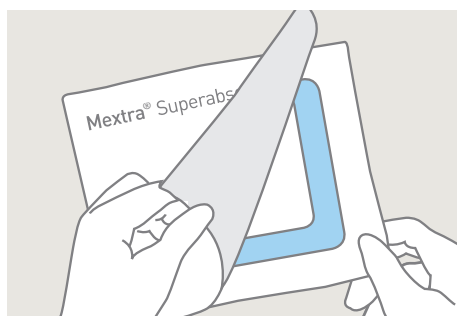


1. Clean the wound area and select a suitable size of Mextra® Superabsorbent.

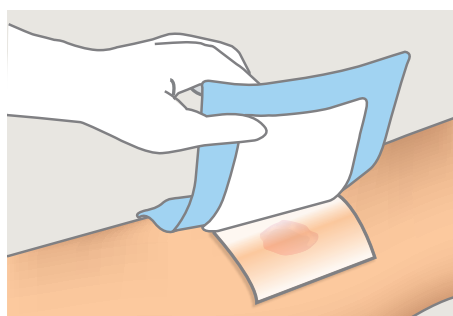

2. Apply Mextra Superabsorbent directly on the wound area with the white side of the dressing facing the wound. For best results, the pad should overlap the dry surrounding skin by at least 2 cm. Mextra Superabsorbent can be used together with primary dressings such as Mepitel®, Mepitel® One or Melgisorb® Plus.

Can be used on chronic wounds, such as pressure ulcers and leg ulcers.

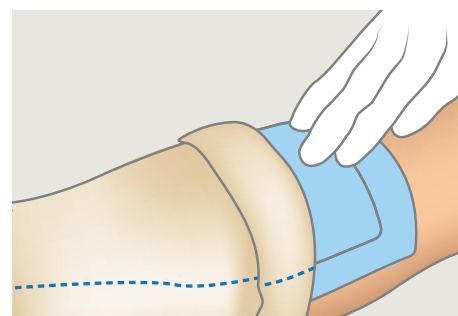

3. Secure Mextra Superabsorbent with a suitable bandage, such as Tubifast® bandage or a fixation tape. It can also be used under a compression bandage where appropriate.

## Key benefits of Mextra Superabsorbent

- Reduces the risk of maceration (protects periwound skin)
- Can be used on venous leg ulcers under compression bandage
- Absorbs and retains exudate containing bacteria
- Contributes to patient comfort during wear
- Easy to apply

## Areas of use

Mextra® Superabsorbent is intended for use on moderately to heavily exuding wounds. Mextra Superabsorbent absorbs wound exudate through the wound contact layer, retains the exudate within the core and thereby reduces the risk of maceration. The water repellent and breathable backing material protects against fluid strike through, which prevents from leakage and helps protect clothing and bed linen.

## Frequency of change

The wound and Mextra Superabsorbent should be inspected regularly. The dressing should be changed based on the condition of the wound or before the dressing has become saturated. Mextra Superabsorbent may be left on the wound for a maximum of seven days.



## Mextra Superabsorbent assortment (Sterile single packed)

Art. No.	Size (cm)	Size (inch)	Pcs/shelf cont.	Pcs/transp cont.
610000	12,5 x 12,5	5 x 5	10	50
610100	12,5 x 17,5	5 x 7	10	40
610200	12,5 x 22,5	5 x 9	10	90
610300	17,5 x 22,5	7 x 9	10	40
610400	22,5 x 27,5	9 x 11	10	40
610500	22,5 x 32,5	9 x 13	10	50
610600	22,5 x 42,5	9 x 17	10	50
610700	10 x 10	4 x 4	10	50
610710	10 x 15	4 x 6	10	40
610720	10 x 20	4 x 8	10	90
610730	15 x 20	6 x 8	10	40
610740	20 x 25	8 x 10	10	40
610750	20 x 30	8 x 12	10	50
610760	20 x 40	8 x 16	10	50

Note: Not all articles are available in every country. Please contact your local Mölnlycke Health Care representative for information about articles available in your country.

Reference: Mölnlycke Health Care. Data on file. 2020.

Find out more at [www.molnlycke.com](http://www.molnlycke.com)

Mölnlycke Health Care AB, P.O. Box 13080, Gamlestadsvägen 3 C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00. The Mölnlycke, Mextra, Mepitel, Melgisorb and Tubifast trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2021 Mölnlycke Health Care AB. All rights reserved. HQIM002717

  
**Mölnlycke®**